

CE 0197

Disposable Silicone Foley Catheter

Instructions for Use




Guangdong Baihe Medical Technology Co., Ltd
-Foshan, Guangdong, P.R. China

File No.: IFU-11-01 Ver. 2019B
Effective Date: 2019-07-17

Intended Use

Disposable Silicone Foley Catheter is a thin,sterile tube inserted into the bladder to drain urine.

Specification

	6Fr, 8Fr,10Fr, 12Fr, 14Fr, 16Fr,18Fr
	6Fr, 8Fr,10Fr, 12Fr, 14Fr,16Fr,18Fr,20Fr, 22Fr,24Fr,26Fr
	16Fr,18Fr,20Fr, 22Fr,24Fr,26Fr

Method of Usage

1. Peel off the single package of catheter,confirm the catheter is applicable.
2. Generously lubricate tip and shaft of the catheter..
3. Carefully insert catheter tip into the bladder(normally indicated by urine draining flow),and then a further 3cm to ensure that balloon also insert urinary bladder .
4. Use a syringe without needle,inflate balloon with distilled water or saline.
5. For deflation ,cut off inflation funnel,above the valve or using a syringe without needle push into valve to facilitate drainage.










Contraindications:

No absolute contraindication.

Notice

1. Do not use if the package has been previously opened or damaged.
2. Can not be reused, or likely cause cross-infection.
3. Every access if the store room should keep clean and sterile.
4. The validity sterilization period is 36 months.

Cautions

 Read instructions before use	 Single use only	 Expired date
 Do not use if package is damaged	 Manufacturer	 Batch Number
 Sterilized by ethylene oxide	 EC Representative	 Product Number



GUANGDONG BAIHE MEDICAL TECHNOLOGY CO., LTD.

No. 89, Taoyuan East Road, Nanhai, Foshan, Guangdong Province, P.R.C.

Tel: +86-757-81207300 +86-757-81207302 +86-139-28979836

Fax: +86-757-81207311

Email: sales@baihemedical.com

Website: www.baihemedical.com

